




German Medical Gas Outlets

Neo Series Flat-faced Pressure Lid Outlets

- Color-coded for easy gas service identification
- Modular structure, convenient maintenance
- German (DIN) Gas Outlet is CE listed and compliant with EN ISO 9170-1 and DIN 13620-2


Specifications

GENTEC® gas outlets are designed for delivery of medical gas from a central pipeline system. These outlets are 100% individually tested for leaks to ensure safety. The outlets are designed with safety in mind and comply fully with ISO 9170-1. Outlets are manufactured in an ISO 9001: 2008 and ISO 13485 registered facility.

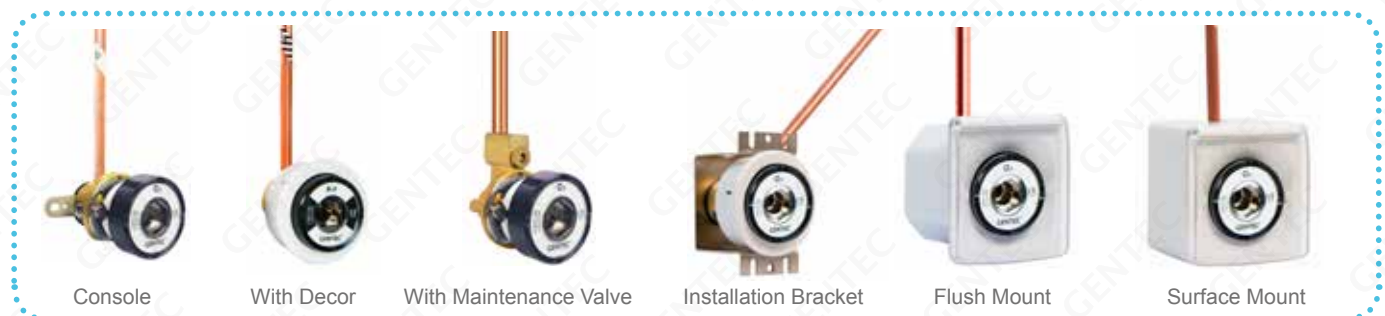


Dimensions units in millimeter(s)

Side

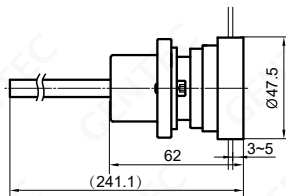


180° Tubing

Side

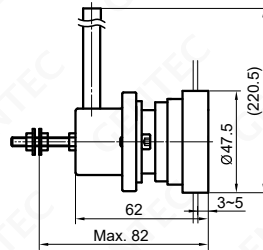


90° Tubing


Back

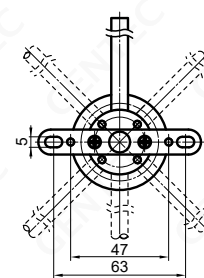


90° Tubing with Console Plate

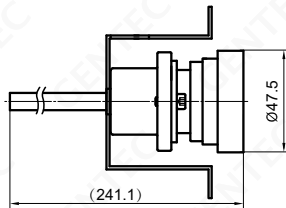


180° Tubing with Installation Bracket

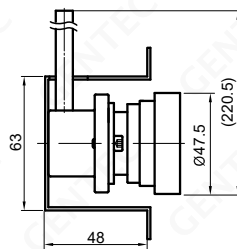


90° Tubing with Installation Bracket

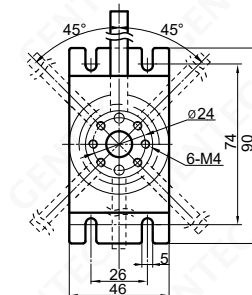


90° Tubing with Installation Bracket

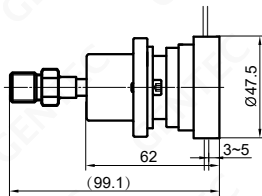


180° M14 Thread

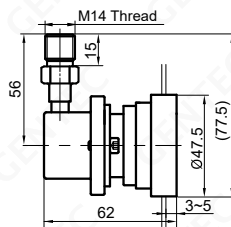


90° M14 Thread

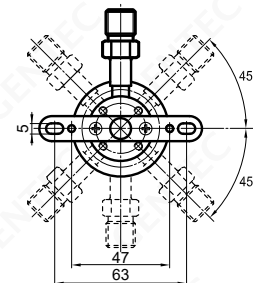


90° M14 Thread with Console Plate

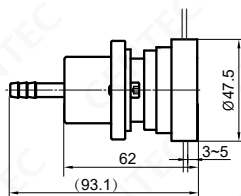


180° Hose Barb

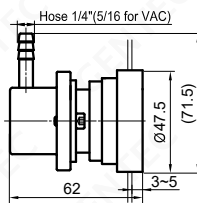


90° Hose Barb

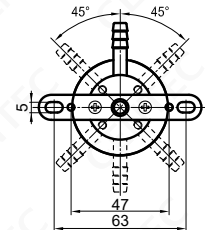


90° Hose Barb with Console Plate

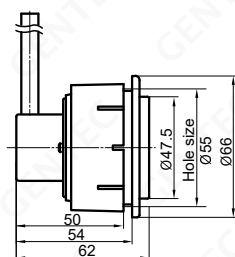

90° Tubing with Square Decor

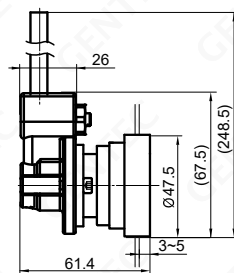


90° Tubing with Maintenance Valve

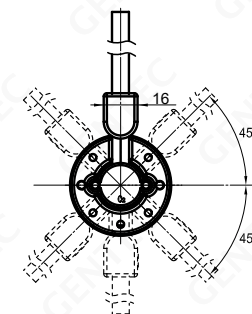


90° Tubing with Maintenance Valve

Dimensions units in millimeter(s)

Surface Mount

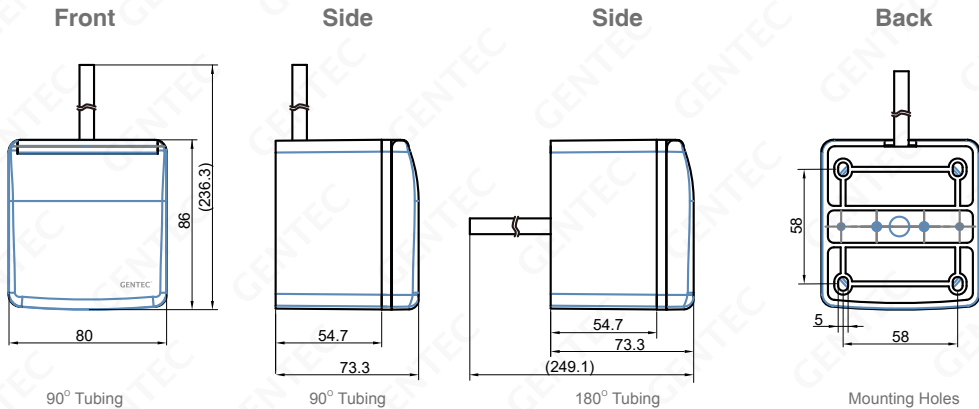

Flush Mount

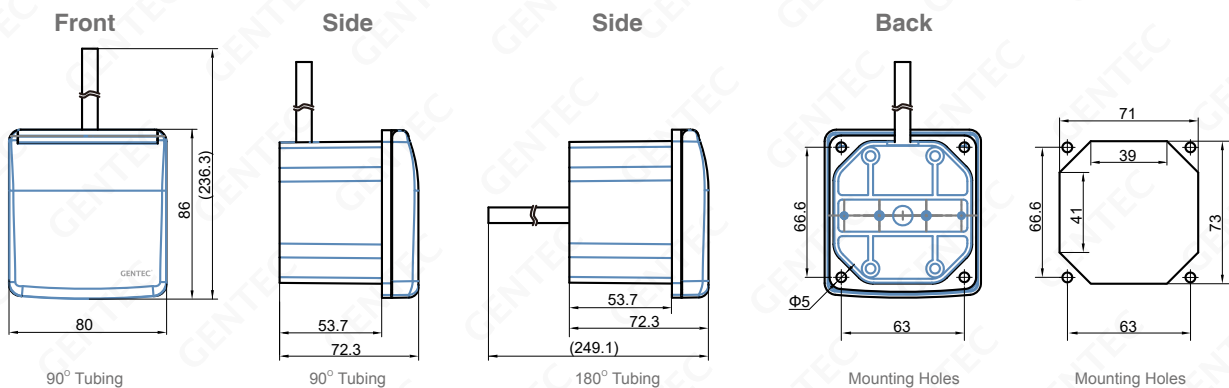

Repair Schematics



1. Pull the pressure lid off



1A. Pull the pressure lid and the decor off



2. Loosen the two retaining bolts 2-3 mm
Pull out the body at an angle



3. Replace the sealing ring without disassembling
the panel mount

Ordering Information (Console Type)

3816E -	V -	S	P8 -	P /	B
Series	Gas Service	Inlet Direction	Inlet Connection	Lid Type	Type of Mounting
3816E: GSO (DIN)	O: Oxygen V: Vacuum A: Air (400 kPa) Z: N ₂ O C: CO ₂ * N: N ₂ -800* A8: Air-800*	S: 90° R: 180° SJ: 90°, with Maintenance valve	P8: 8 mm O.D. Tubing P10: 10 mm O.D. Tubing P12: 12 mm O.D. Tubing H4: 1/4" Hose Barb H5: 5/16" Hose Barb M14: M14 X 1.5 VCR Thread	P: Flat-faced pressure lid P1: Flat-faced pressure lid with square decor	None: With console plate B: With installation bracket

*: Please ask GENTEC or your local distributor for availability.

Type of Mounting	Configuration	Type of Mounting	Configuration
P	 Flat-faced pressure lid, console type with console plate	P / B	 Flat-faced pressure lid, console type with installation bracket
P1	 Flat-faced pressure lid with round decor, console type with console plate	P1 / B	 Flat-faced pressure lid with round decor, console type with installation bracket

Ordering Information (Wall Type)

3826E -	V -	S	P8 -	P /	F
Series	Gas Service	Inlet Direction	Inlet Connection	Lid Type	Type of Mounting
3826E: GSO (DIN)	O: Oxygen V: Vacuum A: Air (400 kPa) 2: N ₂ O C: CO ₂ * N: N ₂ -800* A8: Air-800*	S: 90° R: 180°	P8: 8 mm O.D. Tubing P10: 10 mm O.D. Tubing P12: 12 mm O.D. Tubing H4: 1/4" Hose Barb H5: 5/16" Hose Barb M14: M14 X 1.5 VCR Thread	P: Flat-faced pressure lid P1: Flat-faced pressure lid with square decor	F: Flush Mount W: Surface Mount

*: Please ask GENTEC or your local distributor for availability.

Type of Mounting	Configuration	Type of Mounting	Configuration
P / F	 Flat-faced pressure lid, wall type, flush mount with protective cover	P / W	 Flat-faced pressure lid, wall type, surface mount with protective cover
P1 / F	 Flat-faced pressure lid with square decor, wall type, flush mount	P1 / W	 Flat-faced pressure lid with square decor, wall type, surface mount